

**MYDIAGNOSTICK**



# Atrial Fibrillation Screening Device



# How to hold for screening

**MYDIAGNOSTICK**



# Basic Software Operation

File Patients Screenings Devices

New Edit Delete View Delete PDF Reassign Prepare Help

Screening ECG MyDiagnostick Help

Name Date Screening ID

The MyDiagnostick device is connected to the Management Studio computer software with the USB cable. The ECGs stored on the MyDiagnostick device are automatically transferred to the software

Interrogate MyDiagnostick

Prepared for: no patient or screening

Measurements: 2

Patient  Screening

New

Reading measurements completed.

Preview Save Discard Cancel

File Patients Screenings Devices

New Edit Delete View Delete PDF Reassign Prepare Help

Screening ECG MyDiagnostick Help

Name	Date	Screening ID
------	------	--------------

Interrogate MyDiagnostick

Prepared for: no patient or screening

Measurements: 2

Patient  Screening


New

Reading measurements completed.

Preview Save Discard Cancel


The **Preview** button is clicked to view the ECGs

Serial number  
20180900084

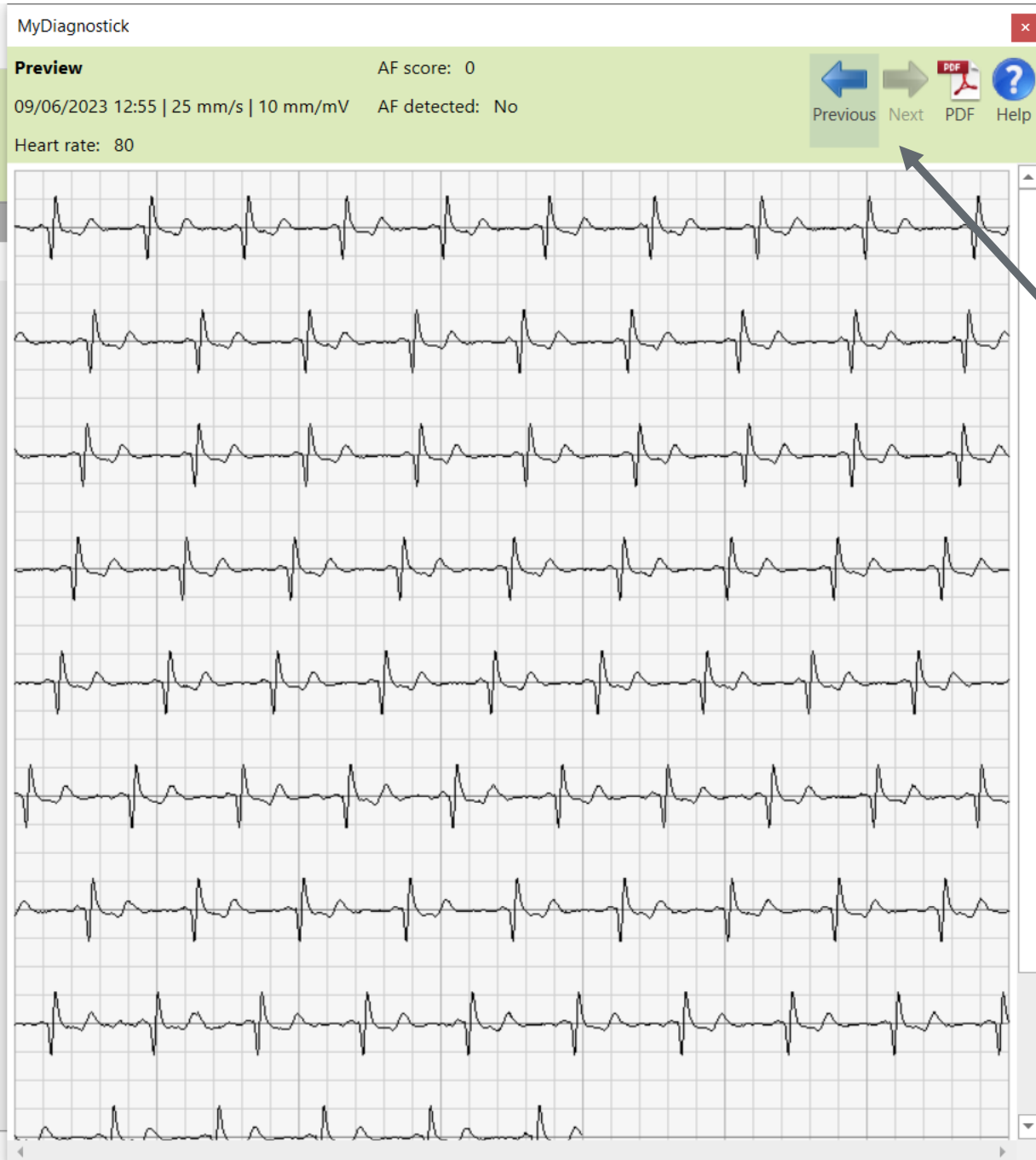


The **Previous** and **Next** buttons are used to view each of the ECGs

Serial number  
20180900084

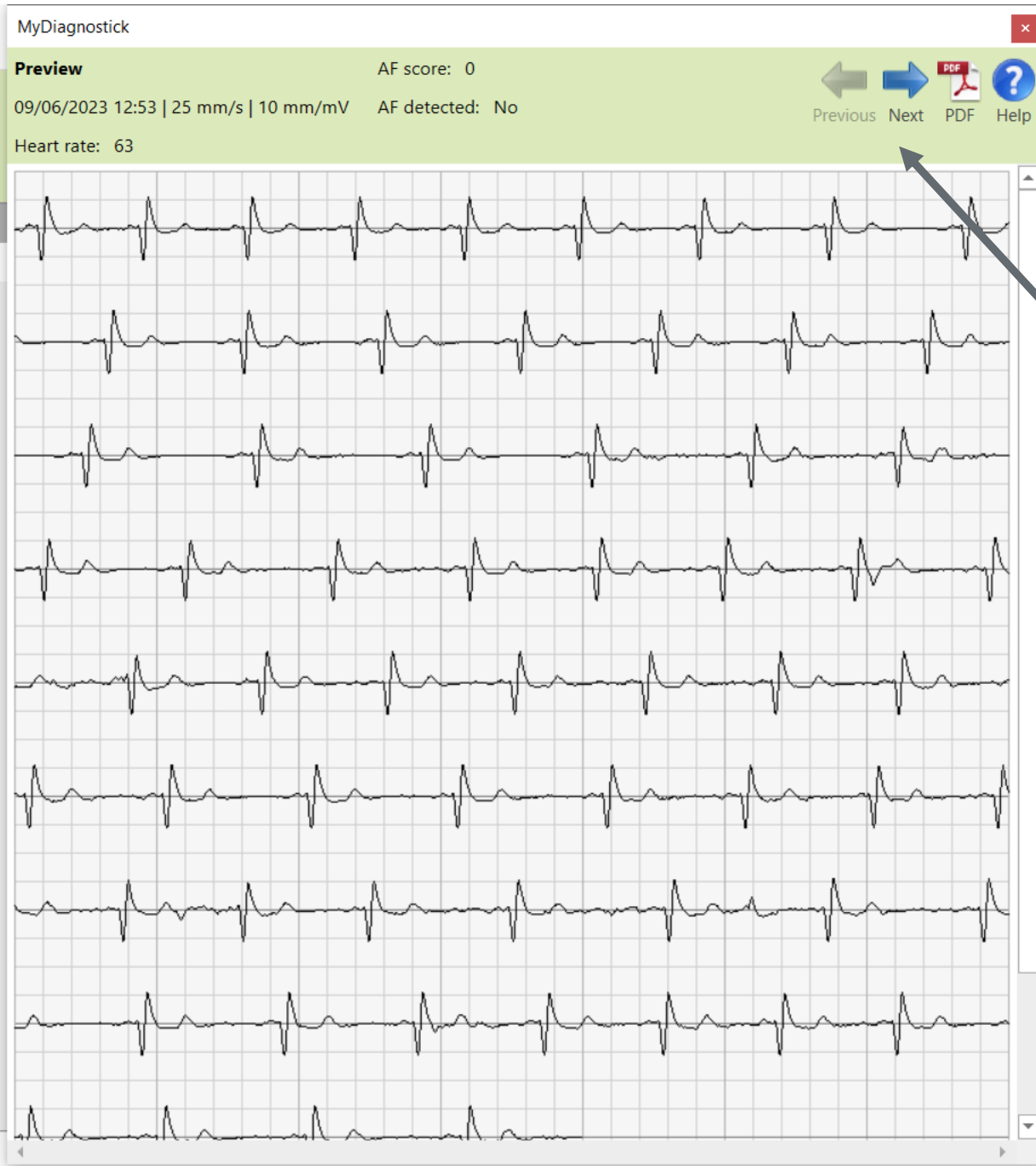

The **Previous** and **Next** buttons are used to view each of the ECGs

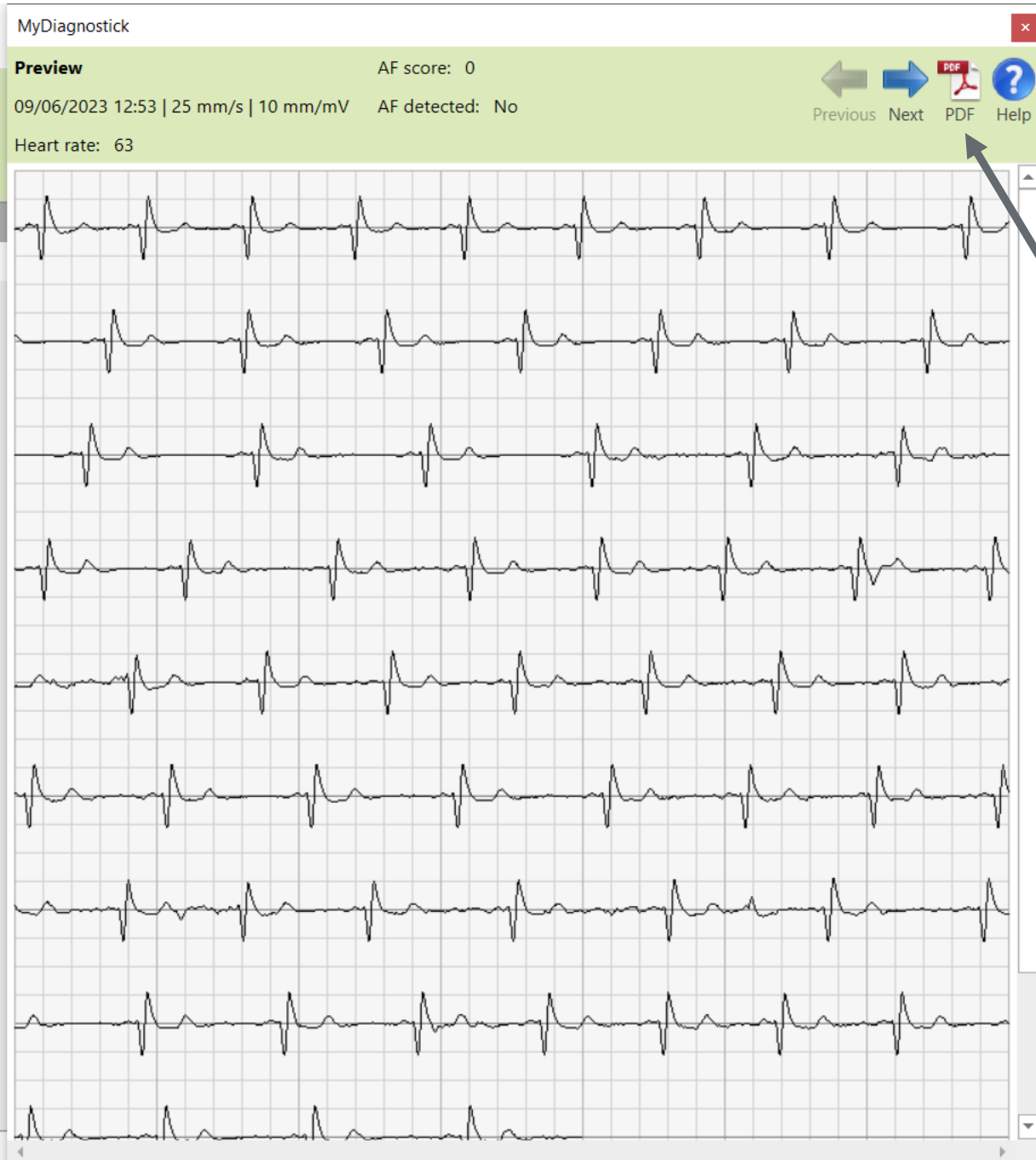


ECG traces can be saved as **PDF** documents and attached to the patient's medical record if required



Heart rate : 63  
AF score : 0  
AF detected : No  
AF confirmed :

Paper speed : 25 mm/s  
Gain : 10 mm/mV  
Recorded : 09/06/2023 12:53  
Serial number : 20180900084

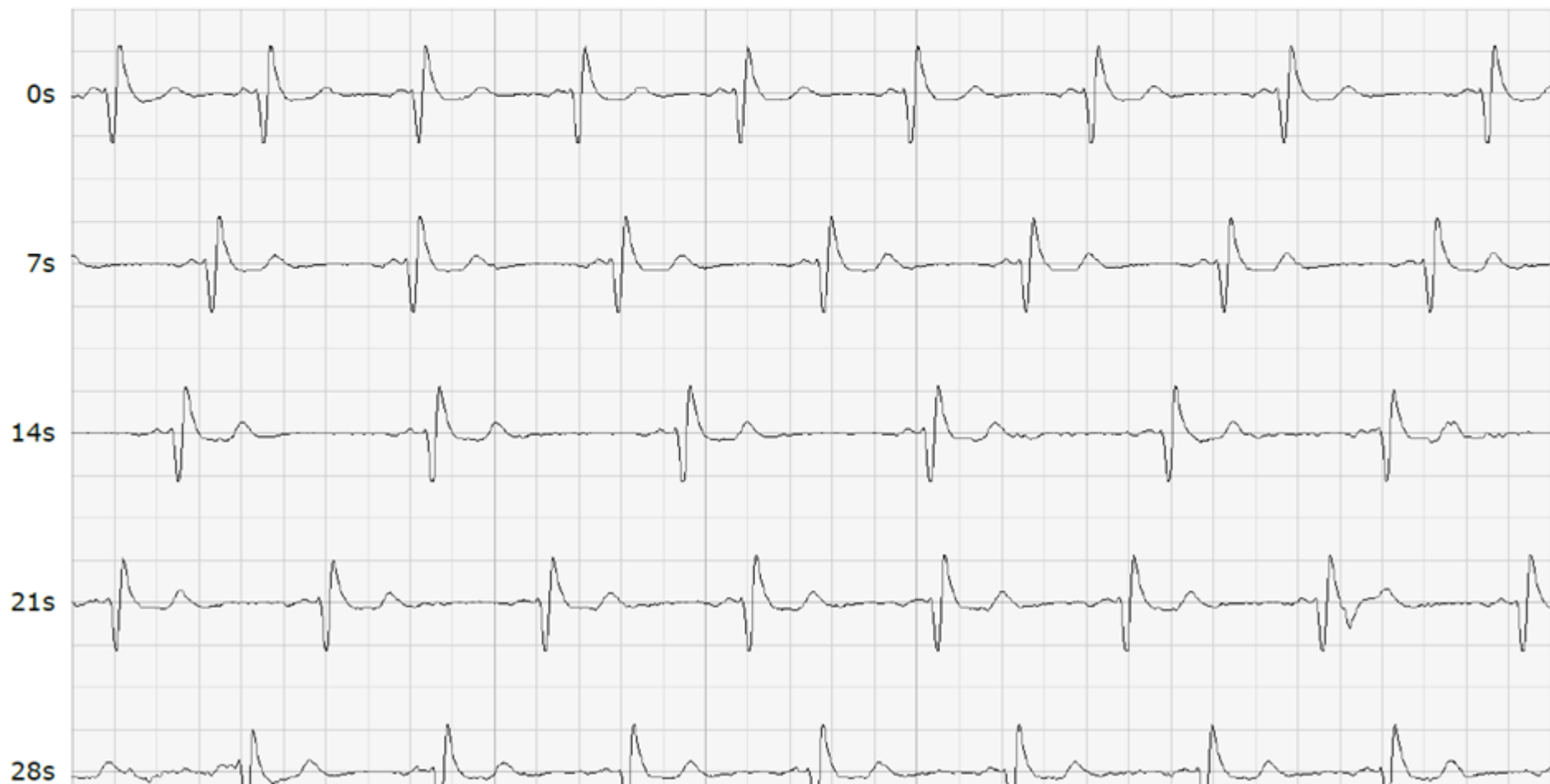

Sample PDF  
ECG report



# MYDIAGNOSTICK



**Tel:** 0114 2433896

**Web:** [numed.co.uk](http://numed.co.uk)

**Email:** [info@numed.co.uk](mailto:info@numed.co.uk)



Maple Ridge, BC 6:00 pm



[www.numed.co.uk](http://www.numed.co.uk)